
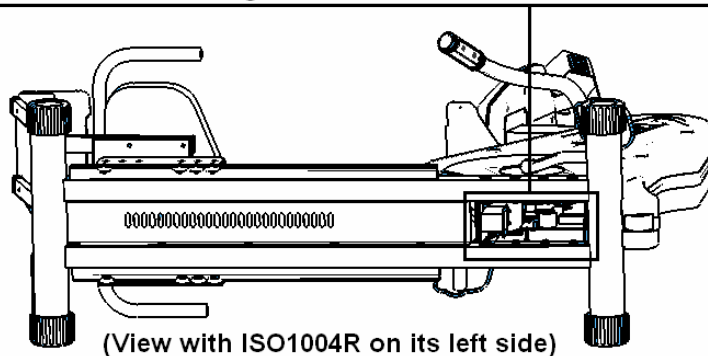


ISO1004R Battery Replacement

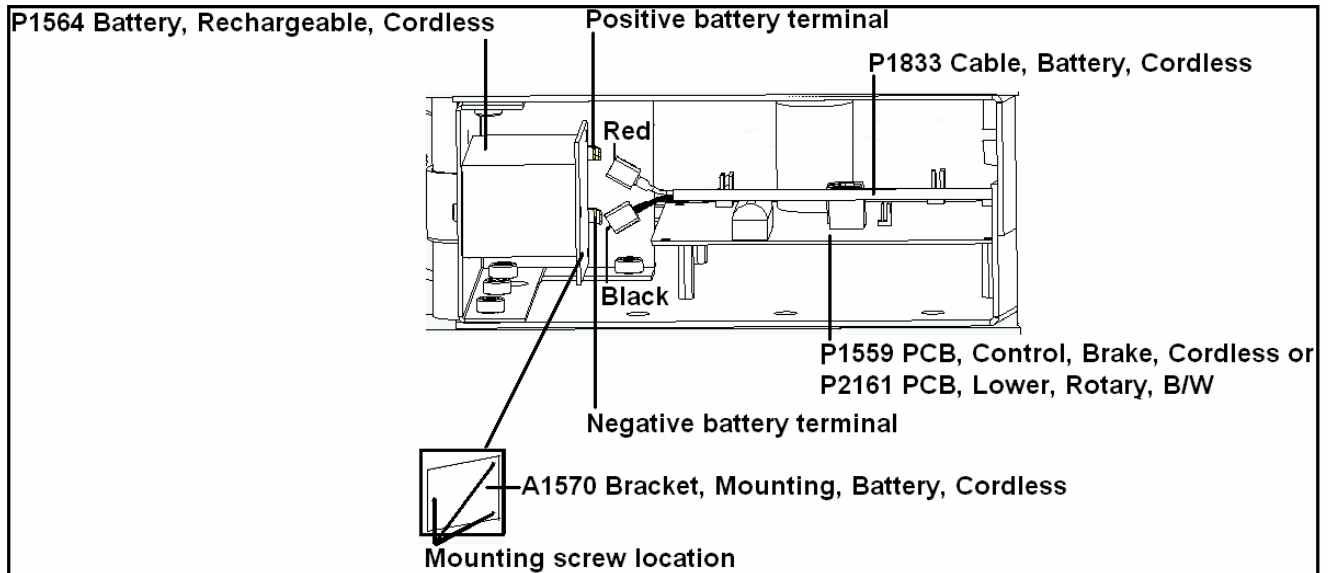

1. Lay the unit on its left side and locate the battery.
2. Remove the three (3) Philips screws running through the battery mounting bracket (A1570).
3. Pull the battery out the bottom of the unit.
4. Disconnect the two (2) wires connected to the battery.
5. Connect the two (2) wires to the new battery. The red wire connects to the positive terminal while the black wire connects to the negative terminal.
6. Install the new battery.
7. Install the battery mounting bracket.
8. Insert the three (3) screws through the battery mounting bracket and tighten down.
9. Verify connections and stand the unit upright.
10. Test unit operations.