


ISO1004R Pedal Removal / Replacement

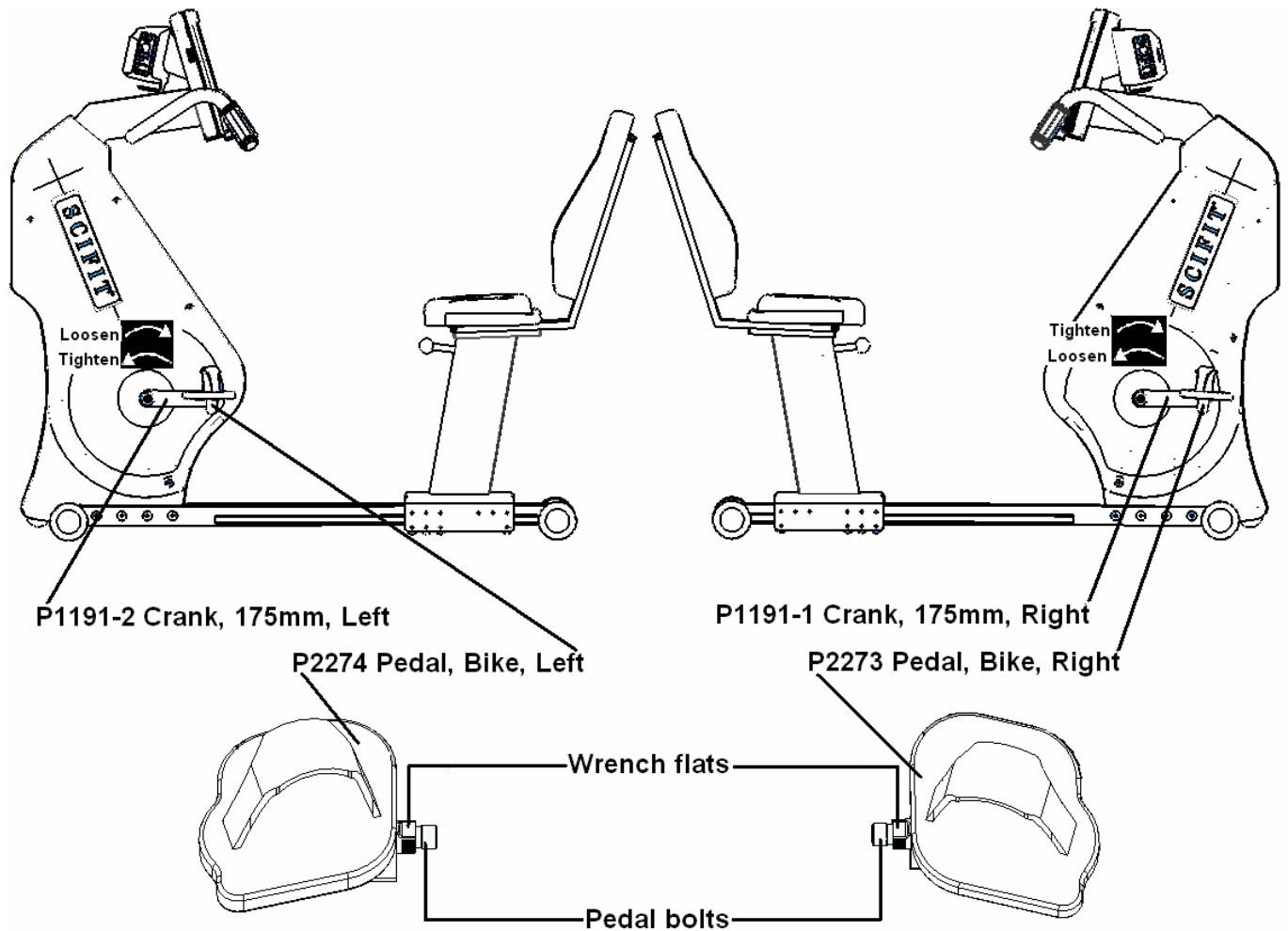

Removal of Left Pedal

1. While looking at the left pedal (P2274) from the left side view (shown above), place either a 15mm or 5/8" open end wrench on the wrench flats on the pedal bolt.
2. Turn the wrench clockwise until the threads on the pedal bolt disengage from the threads of the left crank arm (P1191-2).

Removal of Right Pedal

1. While looking at the right pedal (P2273) from the right side view (shown above), place either a 15mm or 5/8" open end wrench on the wrench flats on the pedal bolt.
2. Turn the wrench counterclockwise until the threads of the pedal bolt disengage from the threads of the right crank arm (P1191-1).

Installation of Left Pedal

1. While looking at the left pedal from the left side view (shown above), start the pedal bolt threads by rotating the pedal bolt counterclockwise, by hand, until you can't tighten any further.
2. Place either a 15mm or 5/8" open end wrench on the wrench flats on the pedal bolt and tighten.

Installation of Right Pedal

1. While looking at the right pedal from the right side view (shown above), start the pedal bolt threads by rotating the pedal bolt clockwise, by hand, until you can't tighten any further.
2. Place either a 15mm or 5/8" open end wrench on the wrench flats on the pedal bolt and tighten.